


# Universal–Single Use XactTrace™ Instructions

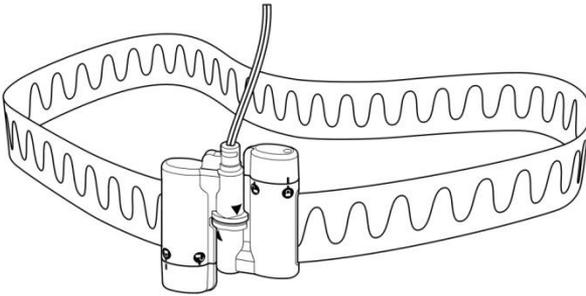

The Universal-Single Use XactTrace Respiratory Inductive Plethysmograph (RIP) sensor belt is a respiratory effort sensor used in the diagnosis of sleep disordered breathing. XactTrace measures changes in inductance, and converts it into a digital signal that gives both qualitative and quantitative data. This sensor is more sensitive and reliable than typical respiratory effort sensors, especially in cases of paradoxical breathing. The Universal-Single Use XactTrace belts consist of a belt roll that can be cut to appropriate lengths for each patient and two Belt Locks for the abdomen and thoracic regions.

## Fitting the XactTrace Belts


---

The XactTrace belts must be custom fit for each patient. When using the XactTrace belts, avoid all unnecessary contact with moisture.

1. Encircle the belt around the patient's chest under the arm to approximate the circumference for the thoracic belt.



- When cutting the belt, reduce its circumference by 10-15cm (4-6 in) so it stretches around the thorax. The belt should fit snugly to prevent slippage during the study. It is important to use sharp scissors for a clean cut.  
The wire should not exceed the end of the belt.
- Secure the cut ends of the belt into the Belt Lock with the blue connector according to the following steps:

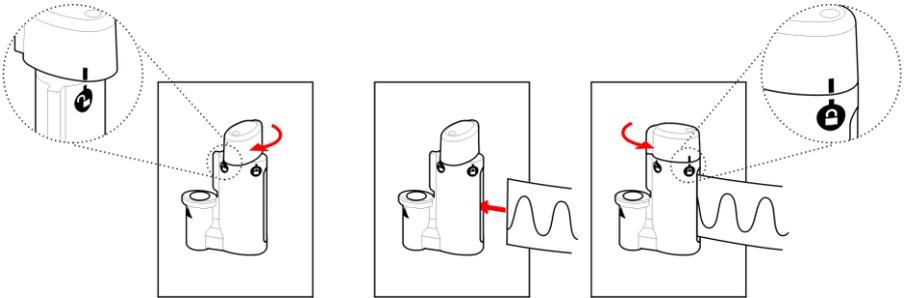

Twist the top end of the Belt Lock clockwise to open the catch. The white mark on the top should match the open lock symbol on the catch.

Insert the cut end of the belt into the catch. Make sure to insert the end all the way to the bottom of the catch.

Twist the top end of the Belt Lock counter-clockwise to close the catch. When the white mark on the top matches the closed lock symbol on the catch, the Belt Lock is properly closed.

- Prepare the abdominal belt in the same manner, only this time, fit the belt around the patient's stomach at the navel, and use the Belt Lock with the yellow connector.


## Attaching the XactTrace Belts

---


The XactTrace belts are intended to be worn over the patient's nightclothes.

- Place the belt with the yellow connector around the patient's stomach, at the navel.

2. Take the two components of the Belt Lock and connect them, as shown below.

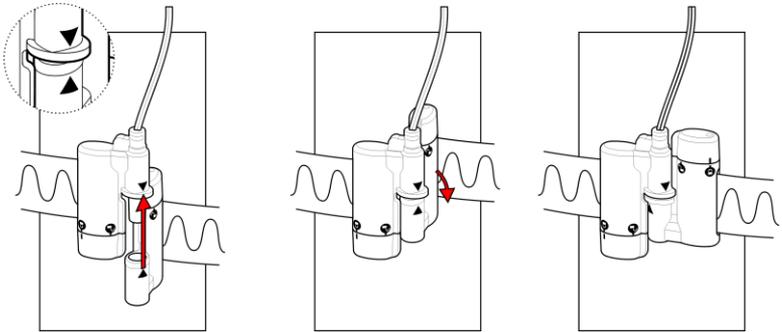

3. Place the second belt with the blue connector around the patient's chest, under the arm.
4. Connect the second Belt Lock as shown above.
5. Insert the connector from the Chest belt into the driver module labeled Thorax (\* in the following drawing).
6. Insert the connector from the Abdomen belt into the driver module labeled Abdomen.

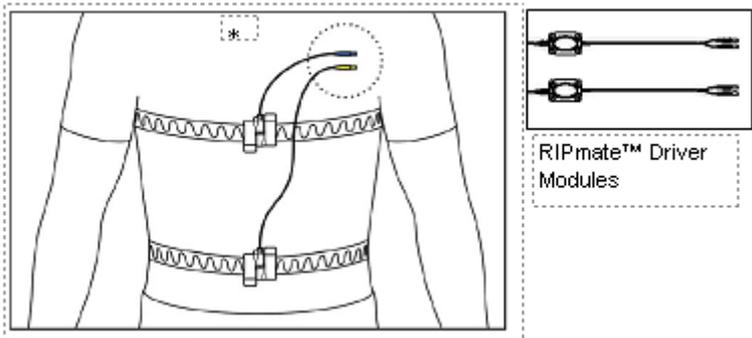

7. Plug the thorax module output into the appropriate bipolar input of your polygraph. The red touch-proof plugs into (+) input, and the black plugs into (-) input.
8. Plug the thorax module output into the appropriate bipolar input of your polygraph.



Note: Do not use two abdomen or two thorax modules in the same recording. This causes interference between the two sensors, resulting in bad signals.

## Storage

---


Proper storage of the XactTrace Belt Locks between recordings is important.

To protect the sensor cable from damage, do not wrap it tightly around the Belt Lock, as it might cause the cable to break where it connects to the Lock.

- Do not wrap the wires around the Belt Locks, as it may cause the wire to break.
- To save battery power, the modules **MUST** be disconnected from the Belt Locks when not in use.



Incorrect

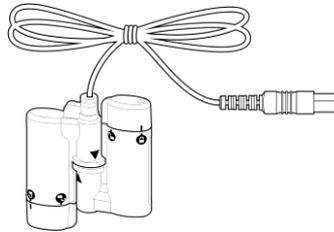

Correct

## Cleaning and Disposal

---

Wipe the Driver Modules and Belt Locks clean with a moist cloth in hospital-grade laundry detergent, then dry them with a clean, dry cloth. Take care to avoid contact of the cleaning solution with the sensor connector and plug of the Belt Lock. The Belt Lock does not require sterilization.

The belts do not need cleaning, as they should be discarded after use.

## Warnings

---

See the list of warnings below.

- Do not stretch the belts too tightly around the patient, as this may cause discomfort. To prevent the belt from slipping during the study, fix the position with medical tape.
- The belts shall be worn over night clothes.
- This product is for diagnostic purposes only, and is not to be used as an apnea monitor.
- Take care not to cut any cables when cutting the belts.
- Do not use damaged sensors or accessories.
- Be certain that cables or sensors do not encircle the patient's neck.
- Do not use the equipment in a magnetic resonance imaging (MRI) environment.
- Do not use the device in an explosive environment—in other words, in the presence of flammable liquids, such as aesthetic mixture with air, or with oxygen or nitrous oxide.
- Connect the XactTrace sensors to an electrically isolated input only. Do not plug the cables into electrical outlets, as this could result in serious electrical shock.
- The device contains a battery; therefore, it must be disposed of properly. Local, state, or national laws might prohibit disposal of batteries in ordinary trash. Contact your local waste authority for information regarding available recycling and disposable options.
- Portable and mobile radio frequency (RF) communications can affect the performance of the device.

## Single Use – Universal XactTrace User Settings

---

User settings are described below.

- Sensitivity – Adjustment of the sensitivity up or down is typically required. Response is dependent upon variables, such as sensor application and patient effort.
- Low Frequency Filter / Time Constant – 0.16 Hz (or 1 second or longer)
- Shorter time constants or higher low frequency filter settings significantly attenuate waveforms.
- High Frequency Filter – 15 to 35 Hz.

# Technical Specifications

See the table below for technical specifications for the Single Use – Universal XactTrace.

<b>Physical Properties</b>	
Belt	Length: Roll of 35 yards, cut to fit each patient. Material: Elastic band with wire.
Cable	Length: 80 inches (205 cm) Material: PVC jacketed; zip-style cord
Module	Box: ABS Cable: PVC jacketed; zip-style cord
<b>Power</b>	
Battery	Non-replaceable lithium coin cell battery. Nominal operation time: 2000 hours (approximately 250 eight hour studies)  The battery is activated once the sensor module is connected to the belt.
<b>Environmental Specifications</b>	
Temperature	Operation: 40°F to 120°F (+5°C to +50°C) Storage: 0°F to 120°F (-18°C to +50°C)
Relative Humidity	Operation: 15 to 95% (non-condensing) Storage: 10 to 95% (non-condensing)
Pressure	Withstands atmospheric pressures from 7.3 psi to 29 psi

### Output Specifications

Output Signal	Maximum signal amplitude: +/- 5mV Frequency Range: 0.2 to 3 Hz Sensitivity: Approximately 50µV/mm
---------------	---

## Certifications

---



The Universal Single-Use XactTrace system is certified to carry the CE mark. The CE mark is a declaration that the Universal XactTrace system is in compliance with the essential requirements set forth by the European Union for medical devices.

## Warranty

---

Embla Systems warrants the sensor to be free of defects in materials and workmanship for a period of one year from the date purchased. The sole liability of Embla Systems and our distributors is limited to replacement or repair of the product at the option of Embla Systems, with no charge for parts or labor if any part is proven to be defective in workmanship, performance, or materials during the warranty period. Under no circumstances shall Embla Systems or our distributors be liable for any loss of revenues or damage, direct, consequential, or incidental, including loss of profit, property damage, or personal injury arising from the use of, or the inability to use this product. This warranty is intended for the original buyer and is in lieu of all other warranties or previous agreements, expressed or implied. This warranty is rendered void if the product is used for other than its intended purpose or is subject to abuse, misuse, tampering, neglect or unauthorized modifications. Use of this product constitutes acceptance of this warranty in total.

## Universal–Single Use XactTrace Instructions

Copyright © 2017 Natus Medical Incorporated. All rights reserved.

Printed in China

**REF** 2870114, Rev. 5

All rights reserved. This manual contains proprietary information, which is protected by copyright and may not be copied in whole or in part except with the prior written permission of Natus Neurology Incorporated. The copyright and the foregoing restrictions on the copyright use extend to all media in which this information is preserved.

This copy of the User Manual shall be used only in accordance with the conditions of sale of Natus Neurology Incorporated or its distributors. Natus Neurology Incorporated makes no representations or warranties of any kind whatsoever with respect to this document. Natus Neurology Incorporated disclaims all liabilities for loss or damage arising out of the possession, sale, or use of this document.

For assistance, please contact Embla Technical Support ([Ottawa.TechSupport@natus.com](mailto:Ottawa.TechSupport@natus.com)).

### MANUFACTURER



Embla Systems  
1 Hines Road, Suite 202  
Kanata, ON K2K 3C7  
Canada  
Tel. : +1.613.254.8877  
Fax : +1.613.270.0627

### EUROPEAN REPRESENTATIVE



Natus Manufacturing Limited  
IDA Business Park, Gort,  
Co. Galway, Ireland  
Tel. : +353-(0)91-64700  
Fax : +353-(0)91-630050

TOLL-FREE IN NORTH AMERICA: 888 NO APNEA (888.662.7632)  
[www.natus.com](http://www.natus.com) [Ottawa.TechSupport@natus.com](mailto:Ottawa.TechSupport@natus.com)



REF: 2870114, Rev. 5