

Please read before using your new Light Drape

## Purpose:

Your new Light Drape is intended for use as a drape to hang from the particular light fixture for which it was fitted. Each drape is labeled with the manufacture and model of the light for which it was designed. Please refer to this label when using the Drape.

## Description:

Light Drapes are made from cotton or cotton blend fabrics and are machine washable.

Each Light Drape is fitted for particular light fixture design and should be used only on that light fixture. Light Drapes are designed to "hug" the light fixture, thus eliminating the need for any additional accessories or adhesives to hold it in place. Light Drapes are designed with flaps that can be individually lifted without removing the drape.

## Other Considerations:

Cautions should be used when determining whether it is appropriate to use a Light Drape by or near any other medical device, such as a heat lamp, bassinet, etc. The fabric of the Light Drape is never to be directly under the source of any heat producing devices.

If in doubt the Light Drape should not be used while that accessory is in use.

## Cleaning:

Light Drapes are made from cotton and cotton-blend fabrics that can be easily laundered. High water temperatures and harsh detergents may cause the fabric dye to fade. Care should be taken to ensure that only color-safe bleach, or other color safe detergents be used to launder your Light Drape.

Chlorine bleach will damage the color and look of your Light Drape. Drying on high temperatures may cause your Light Drape to shrink.

## Instructions for use:

Permanent hook buttons must first be adhered to the outside of your neoBLUE light from which the Light Drape will hang. Four of the hook buttons should be adhered to the outside of the light cover, centering each button on the rounded corner of the light. The additional two hook buttons should be centered along the longer side of the light, adhered to the center front and center back of the light. Buttons should be positioned to just meet and not extend past the bottom edge of the light. Allow the buttons to cure for ten to twelve hours before use.

To hang the drape, align the longer panel with the front of the light. Use light pressure to attach the flap loop tabs to the hook buttons on the light at the center front and corners. Continue to attach the flap loop tabs to all six hook buttons around the light.

 Warning! Confirm vents around the bottom of the light are not blocked and the drape is not hanging down in such a way as it blocks the air flow.

 Warning! Do not use drape in conjunction with radiant warmer.

## Caution:

- 1) Light Drapes are to be used only on those phototherapy lights that allow the accessory. Each drape is labeled with manufacturer and model of the light for which it was designed.
- 2) Fabric from the Light Drape should not touch or interfere with the infant under the light or delivery of the light to the infant.
- 3) A Light Drape should not be used if the use of it interferes with the need for visual monitoring of the infant.
- 4) Prewash each Light Drape prior to first use. Fabrics may contain fabric finishes added by the manufacturer. Normal machine washing and drying will effectively remove these finishes.

## Storage:

Light Drapes that are clean and dry can be folded to store. As with other cotton products, smoothly folding your Light Drapes shortly after removing them from the dryer will help to reduce wrinkles. Light Drapes can be ironed on a cotton setting, if desired.

Do not store Light Drapes in a damp area. If your stored Light Drape has been exposed to moisture, it should be laundered before use.

If you have any questions or concerns about your new product please do not hesitate to contact us.



Natus Medical Incorporated 5900  
First Avenue South Seattle, WA  
98108 USA  
+1 605-802-0400  
www.natus.com



Natus Europe GmbH  
Robert-Koch-Str. 1  
82152 Planegg  
Germany

